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# COLDAMARIS Nasen- und Rachenspray

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## INSTRUCTIONS FOR USE

# COLDAMARIS Nasal and Throat Spray

### What is COLDAMARIS Nasal and Throat Spray?

COLDAMARIS Nasal and Throat Spray is used for moistening the nose and pharynx as well as to support the treatment of cold and allergies.

COLDAMARIS Nasal and Throat Spray is sterile and contains **no preservative**.

### What does COLDAMARIS Nasal and Throat Spray contain?

COLDAMARIS Nasal and Throat Spray is an isotonic sterile solution which contains standardised sea salt (consisting of the ions sodium, potassium, calcium, magnesium, chloride, sulphate, bromide, iodide and hydrogen carbonate) as well as Tyloxapol (wetting agent) and purified water.

### What are the properties of COLDAMARIS Nasal and Throat Spray?

COLDAMARIS keeps the nasal and pharyngeal mucosae moist. COLDAMARIS supports the natural function of the nasal mucosa, particularly in the case of sniffles caused, for example, by cold or allergies.

A dry and irritated nasal or pharyngeal mucosa may result from general air pollution or frequent stays in places with a low relative air humidity such as heated or air-conditioned rooms, cars, trains, or airplanes. Both factors - air pollution and low air humidity - cause drying of the nasal and pharyngeal mucosae.

The filtering function of the nose - that is, its ability to hold back particles in respiratory air (dust, pollen, allergens, and others) - can only be performed optimally by a moist nasal mucosa with a functioning and healthy ciliated epithelium. COLDAMARIS keeps the nasal mucosa moist and thus supports the cleansing and filtering function of the nose.

### Therapeutic Indications - What is COLDAMARIS Nasal and Throat Spray used for?

- For moistening in the case of dry nasal and/or pharyngeal mucosa.
- For cleansing the nose. The wetting agent helps to detach and remove crusts and dried nasal discharge more easily.
- For supporting treatment in the case of a cold, running nose and allergic cold.
- For supplementing treatment when using decongestant medication.

COLDAMARIS Nasal and Throat Spray is suitable for use in children of 1 year and older and in adults.

In infants below 1 year of age, COLDAMARIS should only be used when recommended by a physician.

COLDAMARIS can be used in pregnancy and lactation without hesitation.

### Dosage and Method of administration - How to use COLDAMARIS Nasal and Throat Spray

Several times a day, depending on the individual requirement, spray one or several jets into each nostril or into the pharynx.

The application may be repeated as often as desired and may be prolonged for a longer period of time.

Remove the protective cap before use and pump a few times if necessary until a uniform atomised spray is discharged. COLDAMARIS is now ready for use.

As far as possible, hold the bottle upright when spraying. In any case the suction tube must project into the fluid. If the function of the spray nozzle is impaired the pump mechanism should be operated several times in vertical position until the atomised spray starts to be emitted again. After use, please put the protective cap back on the nozzle.

### When should COLDAMARIS Nasal and Throat Spray not be used?

If you are allergic (hypersensitive) to any of the ingredients of the product.

### Special warnings for safe use

COLDAMARIS is not a medicine! If the condition does not improve or pain occurs consult a doctor.

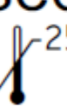
In the event of heavy incrustation or inflammatory processes, be sure to consult a doctor.

For hygienic reasons and to avoid transmission of pathogens, one spray bottle should be used **by one and the same person** only.

Keep out of reach of children!

**Pack size:** Bottle with 20 ml sterile solution, with nebuliser. One spray jet = 0.14 ml

**Shelf-life:** Because of the special filter system the solution remains sterile even after repeated use. Therefore, COLDAMARIS can be used until the date of expiry mentioned on the package.

**Storage instructions:**  Do not store above 25 °C.

**Manufacturer:** MoNo chem-pharm Produkte GmbH, Leystraße 129, 1200 Vienna - Austria

**Distribution:** SIGMAPHARM Arzneimittel GmbH, 1200 Vienna - Austria

 0123

Revision date: July 2019

STERILE A