Siccasan® - Augengel

5 % Dexpanthenol

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5 % Dexpanthenol

Description:

Siccasan® – **Eye Gel** replenishes the eyes natural tear film and protects the ocular surface. **Siccasan®** – **Eye Gel** is a particularly soft and smooth eye gel which distributes quickly over the ocular surface. This rapidly eases the sensation of burning, dry or tired eyes. The eye lids glide over cornea and conjunctiva without resistance.

Siccasan® – **Eye Gel** is suitable for all symptoms of irritated and dry eyes. It provides long lasting relieve from discomfort caused by:

- Environmental stress: sun, wind, cold, smoke, air conditioning and heating
- Visual stress due to, for example, computer screens, television and driving at night
- Wearing of contact lenses
- Advanced age
- Consumption of antidepressants, anxiolytics, hormones and antihistamines

Administration:

For administration in the conjunctival sac of the eye.

Always use **Siccasan® – Eye Gel** according to instructions of this leaflet. If you are not sure, please ask your doctor or pharmacist. If your doctor did not tell you otherwise, follow the dosage mentioned in this leaflet.

Dosage:

Adults:

Administrate 3 – 5 times daily and before going to bed 1 drop gel into the conjunctival sac. Hold the tube upright.

If you are using any additional ophthalmic products, please keep administration at least 15 minutes apart. **Siccasan® – Eye Gel** should always be used last.

Contact lenses should be removed before administering **Siccasan® – Eye Gel** and only be replaced 15 minutes later.

Composition:

5 % dexpanthenol, cetrimide, sodium edetate, carbomer, sodium hydroxide, water for injections

Package size:

1 tube with 10 g

Storage:

Siccasan® – Eye Gel should be stored at room temperature (15-25 °C). Discard 6 weeks after initial use. Do not use after expiry date. Please keep Siccasan® – Eye Gel out of reach of children.

Manufacturer:

AGEPHA Pharma s.r.o., Dial'nicna cesta 5, 90301 Senec, Slovak Republic www.agepha.com, office@agepha.com

Information as of:

July 2017